

NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: Winner Medical Co., Ltd.

Model Tested: WN-N95FGIN

Date Tested: June 10, 2020

These findings pertain to the Winner Medical Co., Ltd., model WN-N95FGIN. The packaging for this product indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Thirty respirators were submitted for evaluation. The respirators were sampled into groups of ten for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found [here](#).

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 99.58% and 99.05%, respectively. All thirty respirators measured more than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

This product has head bands/straps. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirator's represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for [Crisis Capacity Strategies \(during known shortages\)](#).

Evaluation of International Respirators

Test: Modified TEB-APR-STP-0059

Date Tested: June 10, 2020

Report Prepared: June 10, 2020

Manufacturer: Winner Medical Co., Ltd.

Item Tested: WN-N95FGIN (Sample Group 1 of 3)

Country of Certification: China (GB2626-2006)

Pictures have been added to the end of this report.

Filter	Flow Rate (Lpm)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency
1	85	10.5	0.45	0.45	99.55
2	85	10.4	0.57	0.57	99.43
3	85	10.9	0.54	0.54	99.46
4	85	10.4	0.95	0.95	99.05
5	85	10.3	0.52	0.52	99.48
6	85	11.3	0.42	0.42	99.58
7	85	10.8	0.58	0.58	99.42
8	85	10.4	0.91	0.91	99.09
9	85	10.5	0.47	0.47	99.53
10	85	10.9	0.57	0.57	99.43
Minimum Filter Efficiency: 99.05			Maximum Filter Efficiency: 99.58		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

NPPTL COVID-19 Response: International Respirator Assessment

Test: Modified TEB-APR-STP-0059

Date Tested: June 10, 2020

Report Prepared: June 10, 2020

Manufacturer: Winner Medical Co., Ltd.

Item Tested: WN-N95FGIN (Sample Group 2 of 3)

Country of Certification: China (GB2626-2006)

Filter	Flow Rate (Lpm)	Initial Filter Resistance (mmH₂O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency
11	85	10.8	0.45	0.45	99.55
12	85	11.7	0.52	0.52	99.48
13	85	10.9	0.58	0.58	99.42
14	85	10.5	0.51	0.51	99.49
15	85	11.4	0.53	0.53	99.47
16	85	11.7	0.51	0.51	99.49
17	85	10.3	0.55	0.55	99.45
18	85	11.5	0.76	0.76	99.24
19	85	11.3	0.65	0.65	99.35
20	85	10.9	0.54	0.54	99.46
Minimum Filter Efficiency: 99.24			Maximum Filter Efficiency: 99.55		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

NPPTL COVID-19 Response: International Respirator Assessment

Test: Modified TEB-APR-STP-0059

Date Tested: June 10, 2020

Report Prepared: June 10, 2020


Manufacturer: Winner Medical Co., Ltd.


Item Tested: WN-N95FGIN (Sample Group 3 of 3)

Country of Certification: China (GB2626-2006)

Filter	Flow Rate (Lpm)	Initial Filter Resistance (mmH₂O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency
21	85	12.0	0.53	0.53	99.47
22	85	10.8	0.64	0.64	99.36
23	85	11.9	0.49	0.49	99.51
24	85	11.3	0.52	0.52	99.48
25	85	11.0	0.46	0.46	99.54
26	85	10.7	0.53	0.53	99.47
27	85	12.3	0.42	0.42	99.58
28	85	11.5	0.62	0.62	99.38
29	85	11.4	0.53	0.53	99.47
30	85	12.8	0.57	0.57	99.43
Minimum Filter Efficiency: 99.36			Maximum Filter Efficiency: 99.58		


- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

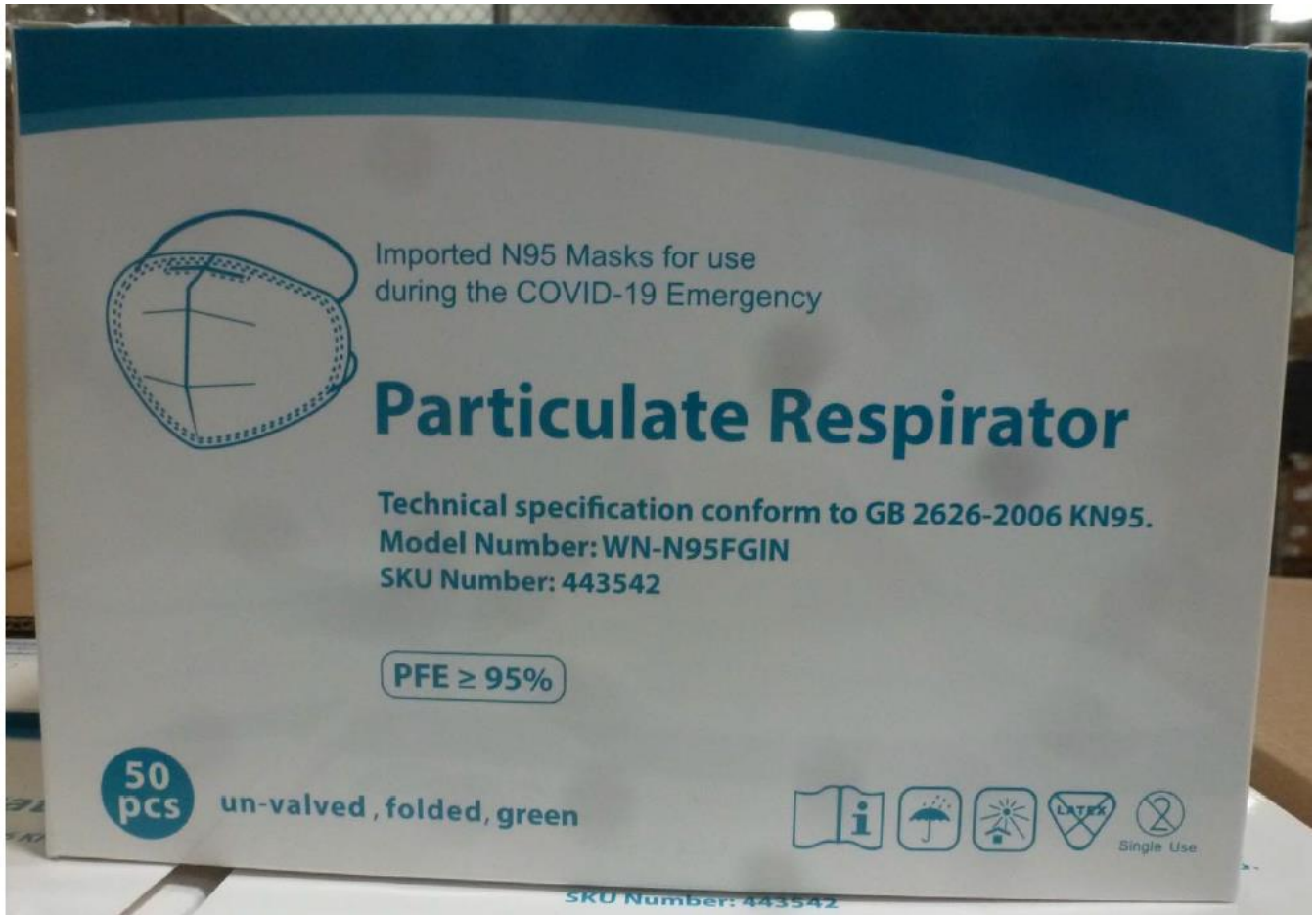


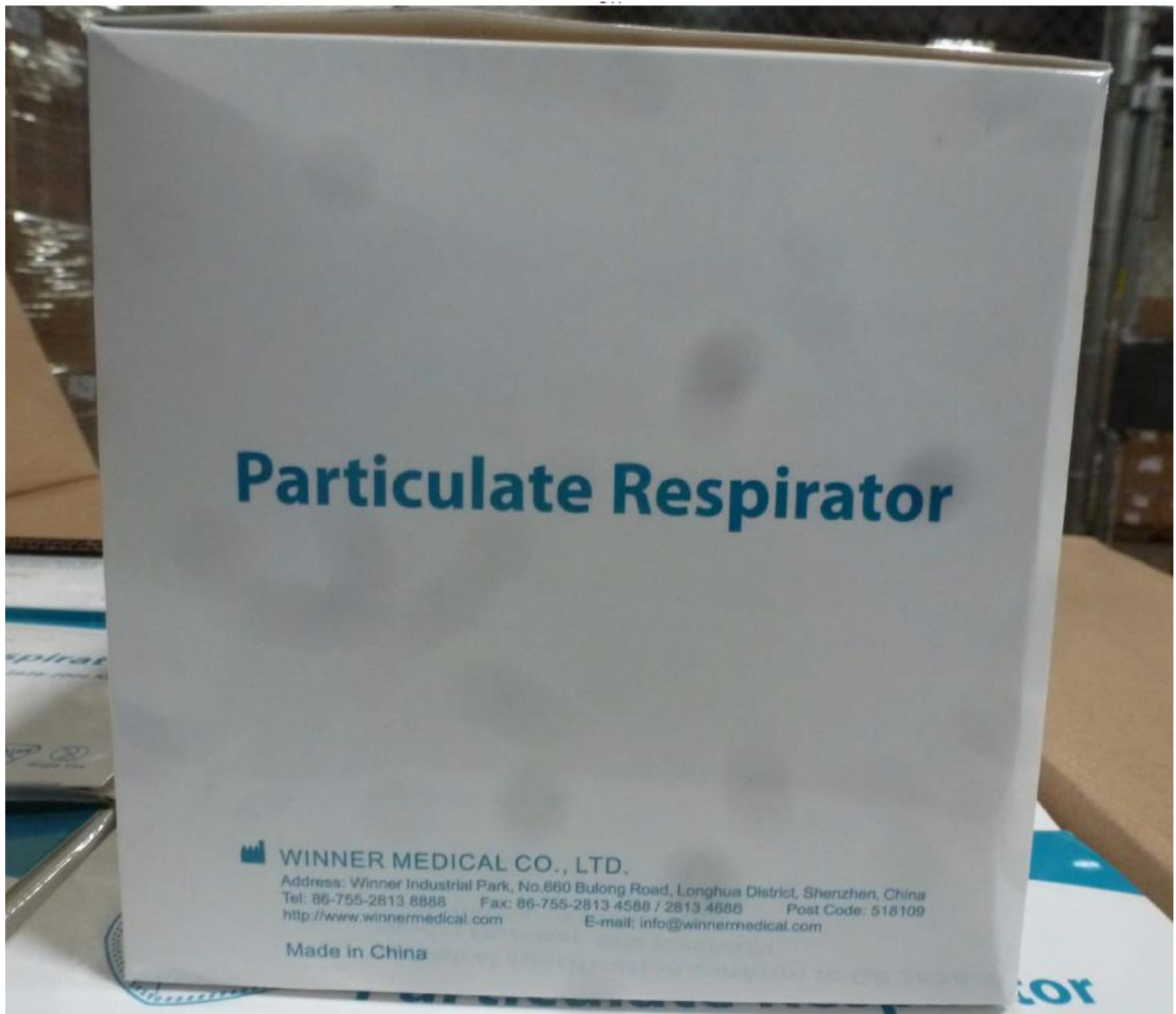




Particulate Respirator

 2020-04-30

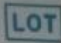 200430-2

 2023-04-29



(01) 1 6941094 02165 0

(17) 230429

(10) 200430-2

winner 合格证

产品名称：防护口罩

型号规格：KN95 折叠式 头带式

Model Number: WN-N95FGIN

产品成分：无纺布50%、热风棉25%、熔喷布25%

符合标准：GB 2626-2006

生产批号：200430-2

生产日期：2020-04-30

使用期限：2023-04-29





【生产企业、售后服务单位】：稳健医疗用品股份有限公司

【住所、生产地址】：深圳市龙华新区布龙路660号稳健工业园

电话：0755-28138888 传真：0755-28134588 28134688 邮编：518109

<http://www.winnermedical.cn> E-mail: info@winnermedical.com

NPPTL COVID-19 Response: International Respirator Assessment

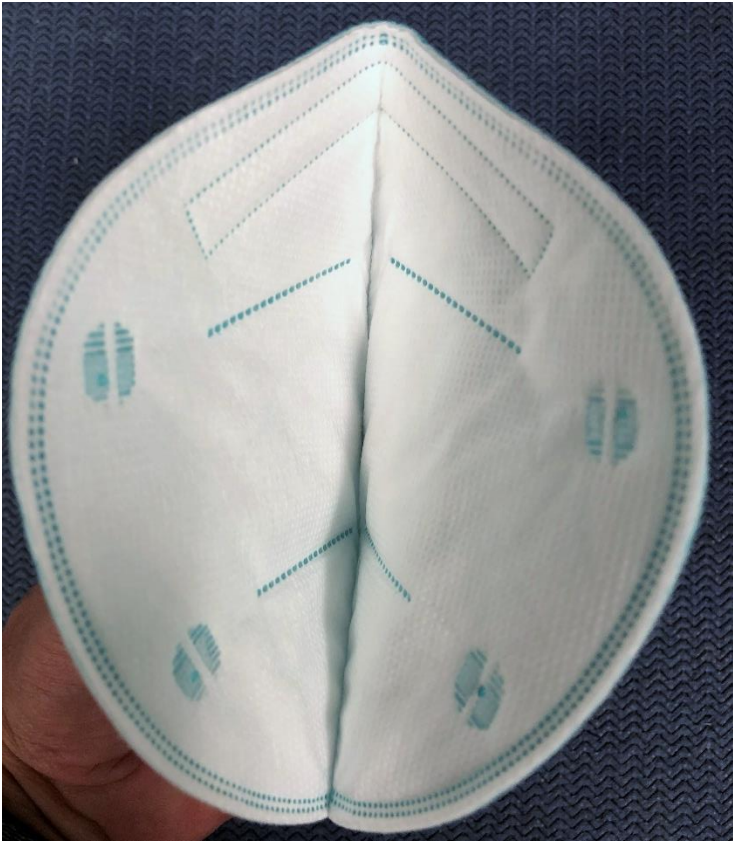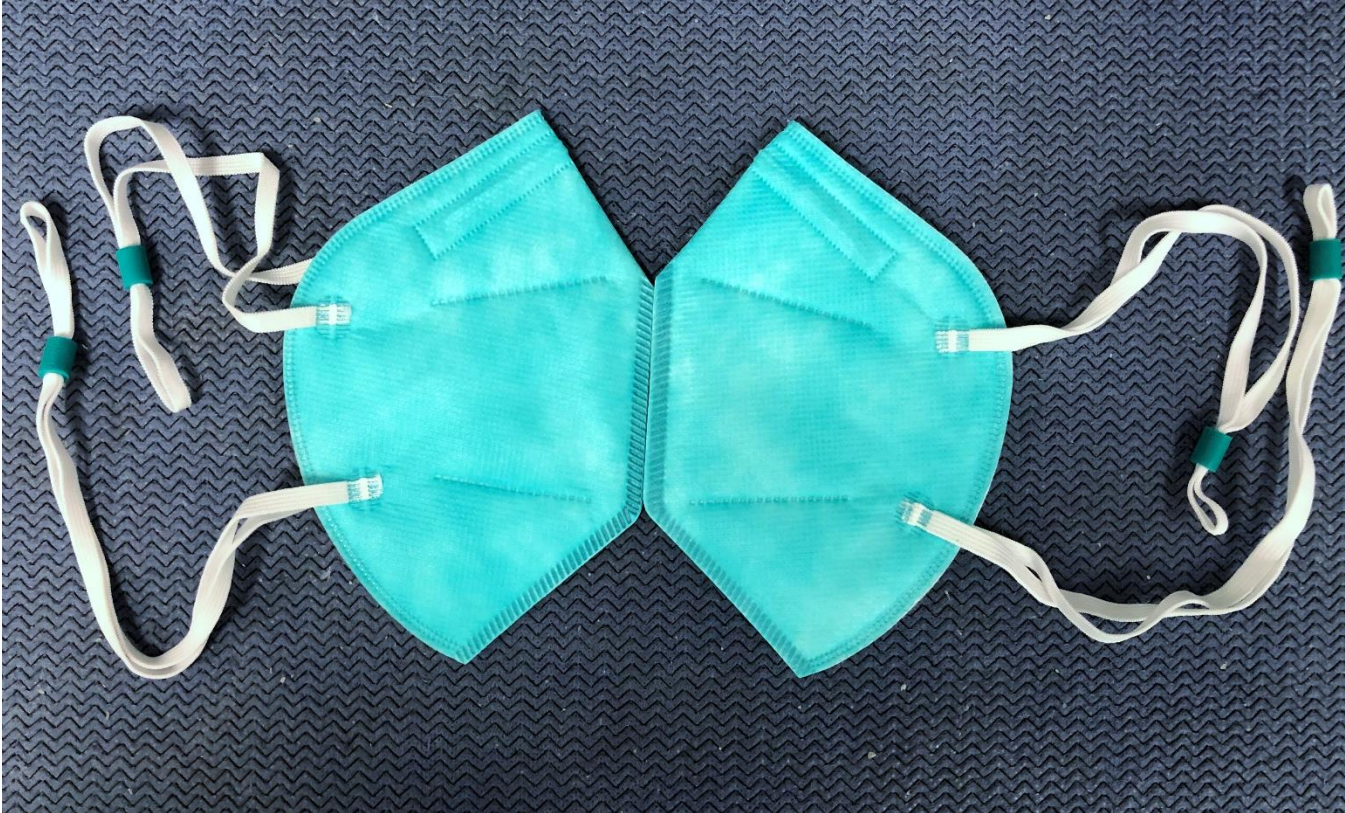

NPPTL COVID-19 Response: International Respirator Assessment



NPPTL COVID-19 Response: International Respirator Assessment



NPPTL COVID-19 Response: International Respirator Assessment

